

STORAGE AND DISPOSAL
 Do not contaminate water, food or feed by storage or disposal.
Pesticide Storage: Store in original container in a cool, well ventilated, locked storage area. Tightly close lid between uses to retain moisture. **Pesticide Disposal:** Dispose of wipe in trash after use. Do not flush. **Container Handling:** Non-refillable container. Do not reuse or refill this container. Discard container and put in trash or offer for recycling if available.


Manufactured for:
Hu-Friedy Mfg. Co., LLC
 3232 N. Rockwell St.
 Chicago, IL 60618 USA
Telephone:
 1-800-Hu-Friedy or 1-773-975-6100

M60051-71345B-0819
 N2458/0819

This product kills the following organisms in 1 minute on pre-cleaned, hard, non-porous surfaces at room temperature:

| | | |
|--|---|--|
| Methicillin-Resistant <i>Staphylococcus aureus</i> (MRSA) | <i>Pseudomonas aeruginosa</i> ATCC 15442 | Carbapenem Resistant <i>Escherichia coli</i> |
| <i>Staphylococcus aureus</i> | <i>Pseudomonas aeruginosa</i> ATCC BAA-2108 | <i>Trichophyton interdigitale</i> (ringworm) |
| <i>Staphylococcus aureus</i> Rosenbach | <i>Salmonella enterica</i> | Herpes Simplex Virus Type 2 |
| Vancomycin-Resistant <i>Enterococcus faecalis</i> (VRE) | Carbapenem Resistant <i>Klebsiella pneumoniae</i> | Herpes Simplex Virus Type 1 |
| Vancomycin Intermediate <i>Staphylococcus aureus</i> (VISA) | <i>Klebsiella pneumoniae</i> | Influenza A Virus (H3N2) |
| Methicillin Resistant <i>Staphylococcus epidermidis</i> (MRSE) | <i>Bordetella pertussis</i> | Influenza A Virus (H1N1) |
| <i>Enterococcus faecium</i> | Extended-Spectrum β -lactamase <i>Escherichia coli</i> (ESBL) | Human Coronavirus |
| <i>Acinetobacter baumannii</i> | <i>Enterobacter aerogenes</i> | Rotavirus |
| Multi-Drug Resistant (MDR) <i>Acinetobacter baumannii</i> | <i>Enterobacter cloacae</i> | Norovirus |

† This product kills the following organism in 2 minutes on pre-cleaned, hard, non-porous surfaces at room temperature:
Mycobacterium bovis BCG tuberculosis (TB)

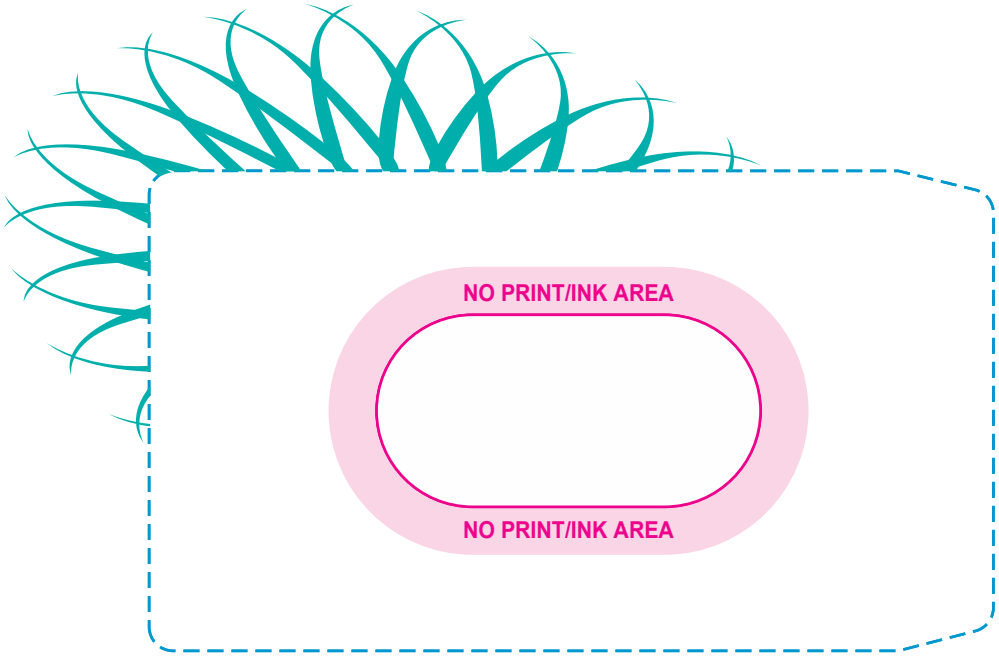

Net Contents: 10 Wipes, 6 x 6.75 inches / Net Weight: 1.6 oz. (45 g)
IMS-2010



DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling.
Advisory Statements: Do not use this product as a baby or skin wipe, or on contact lenses. Do not use on dishes, glasses or utensils. Rinse all surfaces which may come in contact with food with clean potable water after using this product. Not recommended for polished wood or bare wood. Unplug small electrical appliances before cleaning or disinfecting with this product.
Dispensing Directions: Pull back re-sealable label located on top of pack top about 3/4 of the way. Do not completely remove the label from pack. Remove wipe. Reseal with label to prevent evaporation of solution. To prevent evaporation keep pack sealed when not in use.
To Disinfect Hard, Non-porous Surfaces/Articles: Use a fresh wipe to thoroughly wet pre-cleaned hard, non-porous surfaces. Allow surfaces to remain wet for 1 minute prior to wiping or allow surfaces to air dry.

Note: Disinfected surfaces which may come in contact with food must be rinsed with potable water after cleaning and/or disinfection and before reuse.
 † This symbol indicates a 2 minute contact time is required to kill TB.
FIRST AID: If Inhaled: Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably by mouth-to-mouth, if possible. **If on Skin:** Take off contaminated clothing. Immediately rinse skin with plenty of water for 15-20 minutes. **If in Eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. **If Swallowed:** Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to by a poison control center or doctor. Do not give anything to an unconscious person.

Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-535-5053 for emergency medical treatment information.
PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals:
CAUTION: Harmful if swallowed, inhaled or absorbed through skin. Causes moderate eye irritation. Avoid breathing vapors and contact with eyes, skin or clothing. Wear appropriate personal protective equipment based on safe work practices. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.
Physical or Chemical Hazards
Combustible. Do not use or store near heat or open flame.

8.75" W
6"
.75" .625" .625" .75"

STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

Pesticide Storage: Store in original container in a cool, well ventilated, locked storage area. Tightly close lid between uses to retain moisture. **Pesticide Disposal:** Dispose of wipe in trash after use. Do not flush. **Container Handling:** Non-refillable container. Do not reuse or refill this container. Discard container and put in trash or offer for recycling if available.

This product kills the following organisms in 1 minute on pre-cleaned, hard, non-porous surfaces at room temperature:


| | | |
|--|---|--|
| Methicillin-Resistant <i>Staphylococcus aureus</i> (MRSA) | <i>Pseudomonas aeruginosa</i> ATCC 15442 | Carbapenem Resistant <i>Escherichia coli</i> |
| <i>Staphylococcus aureus</i> | <i>Pseudomonas aeruginosa</i> ATCC BAA-2108 | <i>Trichophyton interdigitale</i> (ringworm) |
| <i>Staphylococcus aureus</i> Rosenbach | <i>Salmonella enterica</i> | Herpes Simplex Virus Type 2 |
| Vancomycin-Resistant <i>Enterococcus faecalis</i> (VRE) | Carbapenem Resistant <i>Klebsiella pneumoniae</i> | Herpes Simplex Virus Type 1 |
| Vancomycin Intermediate <i>Staphylococcus aureus</i> (VISA) | <i>Klebsiella pneumoniae</i> | Influenza A Virus (H3N2) |
| Methicillin Resistant <i>Staphylococcus epidermidis</i> (MRSE) | <i>Bordetella pertussis</i> | Influenza A Virus (H1N1) |
| <i>Enterococcus faecium</i> | Extended-Spectrum β -lactamase <i>Escherichia coli</i> (ESBL) | Human Coronavirus |
| <i>Acinetobacter baumannii</i> | <i>Enterobacter aerogenes</i> | Rotavirus |
| Multi-Drug Resistant (MDR) <i>Acinetobacter baumannii</i> | <i>Enterobacter cloacae</i> | Norovirus |

† This product kills the following organism in 2 minutes on pre-cleaned, hard, non-porous surfaces at room temperature:
Mycobacterium bovis BCG tuberculosis (TB)


Manufactured for:
Hu-Friedy Mfg. Co., LLC
3232 N. Rockwell St.
Chicago, IL 60618 USA
Telephone:
1-800-Hu-Friedy or 1-773-975-6100

EPA REG. NO. 70144-4-75372
EPA EST. 50757-WI-003, 50757-WI-001

M60051-713458-0819
N2458/0819




(01) 1 08 89950 10598 3

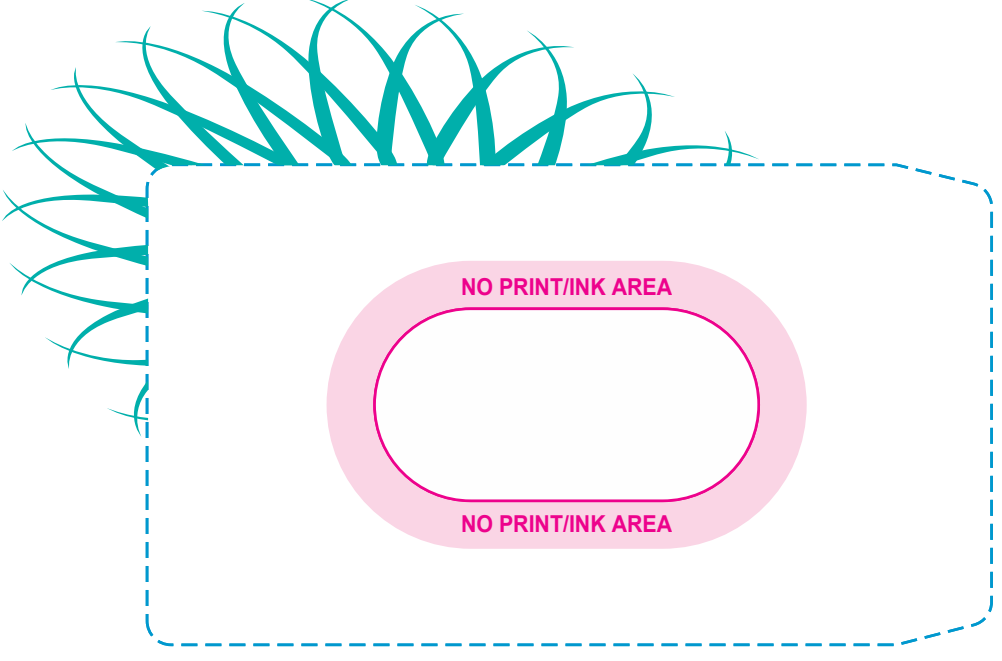

NO PRINT/INK AREA

NO PRINT/INK AREA

Net Contents: 10 Wipes, 6 x 6.75 inches / Net Weight: 1.6 oz. (45 g)
IMS-2010



DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

Advisory Statements: Do not use this product as a baby or skin wipe, or on contact lenses. Do not use on dishes, glasses or utensils. Rinse all surfaces which may come in contact with food with clean potable water after using this product. Not recommended for polished wood or bare wood. Unplug small electrical appliances before cleaning or disinfecting with this product.

Dispensing Directions: Pull back re-sealable label located on top of pack top about 3/4 of the way. Do not completely remove the label from pack. Remove wipe. Reseal with label to prevent evaporation of solution. To prevent evaporation keep pack sealed when not in use.

To Disinfect Hard, Non-porous Surfaces/Articles: Use a fresh wipe to thoroughly wet pre-cleaned hard, non-porous surfaces. Allow surfaces to remain wet for 1 minute† prior to wiping or allow surfaces to air dry.

Note: Disinfected surfaces which may come in contact with food must be rinsed with potable water after cleaning and/or disinfection and before reuse.

† This symbol indicates a 2 minute contact time is required to kill TB.

FIRST AID: If Inhaled: Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably by mouth-to-mouth, if possible. **If on Skin:** Take off contaminated clothing. Immediately rinse skin with plenty of water for 15-20 minutes. **If in Eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. **If Swallowed:** Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to by a poison control center or doctor. Do not give anything to an unconscious person.

Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact **1-800-535-5053** for emergency medical treatment information.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals:

CAUTION: Harmful if swallowed, inhaled or absorbed through skin. Causes moderate eye irritation. Avoid breathing vapors and contact with eyes, skin or clothing. Wear appropriate personal protective equipment based on safe work practices. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.

Physical or Chemical Hazards

Combustible. Do not use or store near heat or open flame.

11" H
4.75" 4.375" 2.5" .375" .375" .375" .375" .75" .75" .75"

4.3975"
.125" .125"

2.5"
.125" .125"

ADVANTACLEAR[®]

Surface Disinfectant Wipes

- 1 Minute[†] Kill Time
- Safe on surfaces such as plastics and non-porous vinyl

Bactericidal / Fungicidal / Virucidal / Tuberculocidal

KEEP OUT OF REACH OF CHILDREN

CAUTION

See back panels for First Aid and additional Precautionary Statements.

ACTIVE INGREDIENTS:

| | |
|--|---------|
| n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈) dimethyl benzyl ammonium chloride | 0.425% |
| n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride | 0.425% |
| Ethanol | 7.750% |
| Isopropanol | 12.250% |

OTHER INGREDIENTS 79.150%

TOTAL 100.000%

VENDOR USE ONLY